

Blade Remover by Delasco
Item: BR-101

An instrument for the safe removal of scalpel blades from handles. Protects hands against contamination by cuts. Stainless Steel and autoclavable. Very easy-to-use.

1. Hold the handle of the scalpel with the blade parallel to the remover.
2. Insert the blade into the blade remover with the post of the handle in the notch that is at the end of the blade remover.
3. Press the jaws of the blade remover, lifting the blade off the handle post. Keep pressure on the blade remover's jaws to keep the removed blade secure.
4. Place the used blade into a sharps container by releasing pressure on the remover jaws.