






608 13th Ave
Council Bluffs, IA 51501

712.323.3269

Delasco.com

DELASCO UNIVERSAL MOBILE STAND: PART NO. 26800

SHOWN WITH OPTIONAL SMOKE EVACUATOR TRAY: PART NO. EV/T

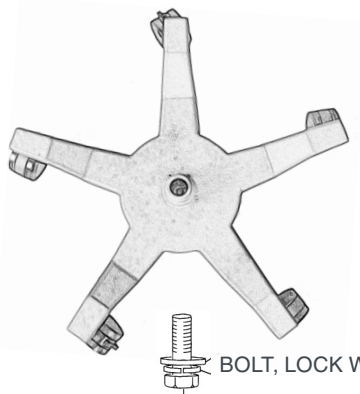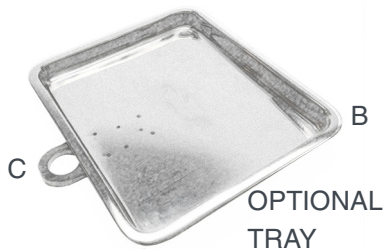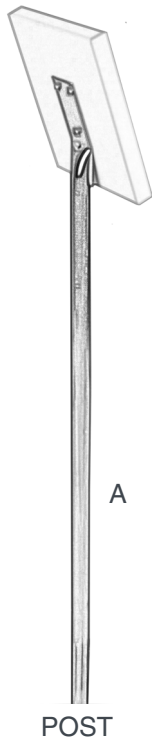

Inspection

Inspect the Universal Mobile Stand upon receipt. If damage has occurred during shipment, do not use. Notify Delasco immediately.

Assembly Instructions

1. Place Optional Tray (B) on Base so that the Tray sits over two of the Base's legs.
2. Position Post (A) through Tray (B) bracket and down into the Base.
3. Tighten small set screw (C) to make sure Tray (B) is held securely in place.
4. Insert Bolt through lock washer, Flat washer and bottom of base. Tighten Bolt to secure Post (A).
5. Use enclosed Hook and Loop straps (included with Optional Tray) to hold cords up and securely against Post (A).

NOTE: The Universal Mobile Stand requires a wall mount kit (sold separately or included with your electrosurgery device) in order to attach the electrosurgical device to the Mobile Stand.

ELEC023(08/18)REVD