



608 13th Ave
Council Bluffs, IA 51501

712.323.3269

Delasco.com

DELASCO 1/16" (0.16CM) DIAMETER REUSABLE ELECTROSURGICAL ELECTRODES

CAUTION: FEDERAL LAW (USA) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN OR HEALTH CARE PROFESSIONAL.

CAUTION: THIS DEVICE IS SUPPLIED NON-STERILE.

CAUTION: DO NOT USE THIS DEVICE IN THE PRESENCE OF FLAMMABLE MATERIALS.

WARNING: READ ALL ELECTROSURGICAL DEVICE INSTRUCTIONS PRIOR TO USING THIS DEVICE.

Inspection and Warranty

Inspect the electrodes upon receipt. If damage has occurred during shipment, do not use and notify Delasco immediately.

Inspection For Use and Reuse

1. Delasco 1/16" Diameter Reusable Electrodes are designed to be used with an electrosurgical handpiece that requires a 1/16" (0.16cm) shaft diameter electrode. Do not attempt to use the electrode if the requirements of the handpiece are for a different diameter size electrode.
2. Prior to use, sterilize the electrode using the instructions supplied below.
3. Insert the electrode into the electrosurgical handpiece. The electrode must be fully and securely inserted in the handpiece for proper function.
4. At the completion of the procedure, remove the electrode from the handpiece and resterilize.

Sterilization Inspection

1. Use aseptic procedures when handling and sterilizing the electrodes.
2. Wipe and remove all debris present on the electrode.
3. Place the electrode in an autoclavable pouch or wrapped cloth.
4. Steam sterilize for a period of 20 minutes at 250°F (121°C), or consult an appropriate nationally recognized standard for sterilizing medical instrumentation. Do not flash autoclave or exceed 250°F.
5. Allow the electrode to cool for 30 minutes prior to use.
6. Inspect the electrode for damage after sterilization. If the electrode has insulation that has deteriorated or cracked, or severe electrode discoloration or damage has occurred, discard the electrode.