

THEIA EYE PATCHES

DIRECTIONS FOR USE

General information:

Theia Eye Patches (Laser, IPL, Derm and Anesthesia) are comfortable, ergonomically designed, latex-free, hypoallergenic, adhesive eye shield containing a patent-pending, inner laser shield and design.

The Laser Eye Patch provides protection from medical lasers with wavelengths between 190 nm – 11,000 nm and require an optical density greater than seven. In addition, the Laser Eye Patch product is certified to protect against any damaging heat transfer or degradation up to five seconds of continual laser radiation

The IPL Eye Patch provides protection from IPL medical devices that operate in the visible and IR portions of the spectrum. It shields the patient from the IPL energy and bright light from these devices

Derm Eye Patch, shields the eye from crystals, creams and other potential debris that may enter the eye during microdermabrasion procedures.

Anesthesia Eye Patch, keeps the patient's eyes closed during anesthesia. This will minimize patient's eye from drying out and keep out debris from entering the eye during the procedure

WARNING: READ ALL INSTRUCTIONS AND WARNINGS BEFORE USING THIS PRODUCT; FAILURE TO FOLLOW THESE INSTRUCTIONS AND WARNINGS MAY RESULT IN ILLNESS, SERIOUS PERSONAL INJURY, OR BLINDNESS.

The instructions stated below for the proper use and application of this product must be read and understood by the person applying all Theia Eye Patches^[MT1]. Failure to read and understand the instructions or failure to follow the instructions may result ^[MT2] in the misapplication of Theia Eye Patches and could lead to serious injury, including blindness in one or both eyes.

Instructions for Use

1. The patient's orbital area must be clean and dry prior to placement^[MT3].
2. The patient must be instructed to lightly close eyelids and refrain from blinking or opening eyes while either eye shield is in place. Failure to keep eyelids closed could deteriorate the adhesion properties which may result in illness, serious personal injury or blindness.
3. Fold liner of eye shield in half

4. Remove one disposable eye shield at a time by pulling the rectangular vertical tab downward from the liner. Do not pull from any other section of the eye shield..
5. Begin placement of the disposable eye shield at the center portion of the orbital area next to the nose bridge while centering the widest part of the eye patch with the middle of the patient's eye.
6. While softly holding eye patch at initial contact, continue placing it around the balance of the orbital area by gently pressing the perimeter (adhesive section) ensuring that all gaps and openings are closed.
7. Repeat Steps 3-5 on opposite eye.

Warning for Laser and IPL Eye Patches.

Prior to use, ensure that the product is suitable for your laser. The wavelength of the laser device MUST match the wavelength range of the Laser Eye Patch.

When using Laser or IPL Eye Patches, a standard light source should be administered to the patient. The patient should confirm that no light could be seen.

DO NOT REMOVE THE THEIA EYE PATCHES UNTIL THE LASER OPERATOR HAS CONFIRMED THAT THE LASER HAS BEEN PLACED IN THE OFF OR STANDBY POSITION

Theia Eye Patch is qualified under the USA Standard ANSI Z136.1 and are marked accordingly with the optical density (OD) Of 7 from 190-11,100nm

Care and cleaning:

When not in use, store in the container the product was supplied in at normal room temperature and not in direct sunlight. These products are intended for one-time use, so do not clean.

WARNING: THEIA EYE PATCHES ARE DISPOSABLE AND ARE NOT INTENDED TO BE REUSED OR REAPPLIED. The reuse or continual replacement of this product could deteriorate its adhesion properties which may result in illness, serious personal injury, or blindness.

FOR RE-ORDERS, QUESTIONS OR ADDITIONAL INFORMATION CONTACT:

THEIA EYE BLOCK
16 Cutler St, Warren, RI 02885
Phone number 888-626-3319
Sales@theiaeye.com