

DELASCO® PRESSURE EARRING

Directions for Use:

1. Screw the two acrylic pieces together loosely using the enclosed plastic screws.
2. Slide the assembled pressure earring over the earlobe, covering the affected area. *See image below.*
3. If the pressure earring is clear, tighten the screws until blanching (whitening) of the skin can be seen. If the skin cannot be seen through the earring, tighten until the device feels snug and secure.* This reduces blood flow to the area.
4. Follow your physician's guidelines for hours of daily wear and duration of use.
5. Clean as needed with soap and water, or rubbing alcohol.



Caution:

*Your first Delasco Pressure Earring should be a clear model so you can see how much pressure causes blanching.

*Discontinue use if you experience pain or discomfort while wearing the Delasco Pressure Earring.

Item #:

Delasco

608 13th Avenue
Council Bluffs, IA 51501
Toll-Free: 1-800-831-6273
Fax: 712-323-1156
questions@delasco.com
www.delasco.com



DFU-DPE(06/18)REV B