

Delasco Acrylic Pressure Earrings

Pressure Earring Styles



DPE-20



DPE-21



DPE-32



DPE-60



DPE-42

Size Comparison Chart - Actual Size*

DPE-20	DPE-21	DPE-32	DPE-60	DPE-42
1.5" length	1.9" length	1.9" length	0.9" length	1" length
0.7" width	0.9" width	0.8" width	0.8" width	1" width

*For an example of how a pressure earring will fit, print this document out at 100%, cut out chosen outline above and position against desired area of ear.

Directions for Use

1. Screw the two acrylic pieces together loosely using the enclosed plastic screws.
2. Slide the assembled pressure earring over the earlobe, covering the affected area. See photos above.
3. If the pressure earring is clear, tighten the screws until blanching (whitening) of the skin can be seen. If the skin cannot be seen through the earring, tighten until the device feels snug and secure.** This reduces blood flow to the area.
4. Follow your physician's guidelines for hours of daily wear and duration of use.
5. Clean as needed with soap and water, or rubbing alcohol.

**Caution:

- Your first Delasco Pressure Earring should be a clear model so you can see how much pressure causes blanching.
- Discontinue use if you experience pain or discomfort while wearing the Delasco Pressure Earring.

Need Help?

✉ questions@delasco.com | ☎ 833.907.1793 | 📍 Delasco, 608 13th Avenue, Council Bluffs, IA 51501